

TEE2⁺

Fact Sheet



A reliable, easy to use meter with bluetooth capability and meets ISO 15197:2015 standards for accuracy¹



- Accuracy to ISO 15197:2015 standards¹
- Bluetooth functionality
- SmartLog compatible helping people manage their condition
- Small blood sample required
- 5 second test time
- Auto coding minimises errors
- Downloads to Diasend
- Stores 1,000 readings (conforming to current DVLA requirements)
- Strip expiration date indicator
- 1, 7, 14, 30, 90 day test averaging tool (total, pre/post meal)
- Large, clear screen

Strip Price: £7.75

Pack of 50
PIP Code: 386-3792

Size 108.4x 55.8 x 15 mm	Weight 70g (with batteries)	Temperature 5-50 °C	Haematocrit 15-65%
------------------------------------	---------------------------------------	-------------------------------	------------------------------

Item	Acceptance Criteria	Result ^{1,2}		
		within ±0.28 mmol/L	within ±0.55 mmol/L	within ±0.83 mmol/L
Glucose Concentration <5.55mmol/L	95% of the measured glucose values falls within ±0.83mmol/L	87/174 (50%)	158/174 (90.8%)	174/174 (100%)
Glucose Concentration ≥5.55mmol/L	95% of the measured glucose values falls within ±15%	within ±5%	within ±10%	within ±15%
		262/426 (61.5%)	394/426 (92.5%)	426/426 (100%)
99% of glucose values fall within zones A and B of the Concensus Error Grid		Within zones A and B 598/600 (99.7%)		

References:

1. TEE2 Plus System Accuracy Test Report, data on file Spirit Healthcare, December 2017.

SPIRIT
HEALTHCARE



24/7 NON-AUTOMATED FREE SUPPORT

0800 881 5423

Spirit House, Saffron Way, Leicester, LE2 6UP

T: 0116 286 5000 E: info@spirit-healthcare.co.uk www.spirit-healthcare.co.uk

Item number: SH/CRSNS/1988/NOV17

Date of preparation: December 2017